

HVE TIP SOLUTIONS FOR DENTAL HYGIENISTS – 3 options

OPTION #1. *Widemouth HVE Tip Insert inserted into a 6" HVE tip that has been cut in half. Use with a short On/Off HVE Valve opposed to standard long valves.*



Widemouth HVE Tip shown inserted onto a vented 6" HVE tip that has been cut in half. Insert Widemouth HVE Tip into vented (slot) end. American Dental Accessories www.amerdental.com Widemouth HVE Tip Insert (Product # 21-792) & Vented HVE tip (Product #21-75)



www.amerdental.com *Short metal lever style high volume evacuator with variable on/off vacuum control and 360°swivel base connector. Autoclavable aluminum valve is easily disassembled for cleaning, but should not be cleaned by sonic methods. (Product # 0303)*

For optimal evacuation, use the Blueboa saliva ejector hose, and hygoformic saliva ejector tip placed on one side of the mouth in conjunction with use of Widemouth HVE tip insert. During use, follow the HVE tip it as close to the Air Polishing tip as possible to capture aerosols.



www.theblueboa.com Blue Boa Saliva ejector hose SKU# 100000 and Orsing Hygoformic saliva ejector SKU #10002

OPTION #2. Use shorter (3.75") Young Vent-O-Vac Vented HVE tips available from Young Dental without the widemouth tip insert since when inserted into this tip, it covers the vents defeating its purpose.



www.youndental.com Young Vent-O-Vac 3.75" HVE tips. (Product # 078210)

NOTE: As described in Option #1, Insert the Vent-O-Vac tip onto the short On/Off HVE Valve, and use in conjunction with the Blueboa solutions.

OPTION #3. Use the Isovac isolation/HVE system from Isolite Systems www.isolitesystems.com. No need to use Blue Boa saliva ejector solutions with this system.

